

Welcome!

Salubrion Seat

Thank you for purchasing your new Salubrion Seat.
The following guide will help you enjoy and care for your Seat.

1 Check

Your Salubrion Seat should have a rubber cap covering the bottom of each leg. The bottom of each covered leg should be slightly soft. DO NOT use the Seat if a foot covering is missing.

2 Setup

Place your Salubrion Seat on a flat level surface that is free from debris or liquid. Make sure that each foot makes firm contact with the surface. Place the included seat cushion on top of the Seat and attach the Velcro straps around the rear support posts.

3 Register and Learn More


Please register your Salubrion Seat at <http://www.salubrion.com/register/>

Visit us online at www.salubrion.com to learn more about your Seat and view the entire line of products from Salubrion.

salubrion™
www.salubrion.com

3 Use

Sit on the ground in front of the Salubrion Seat. Lift yourself back on to the Seat, gently setting your buttocks down on the horizontal seating surface.

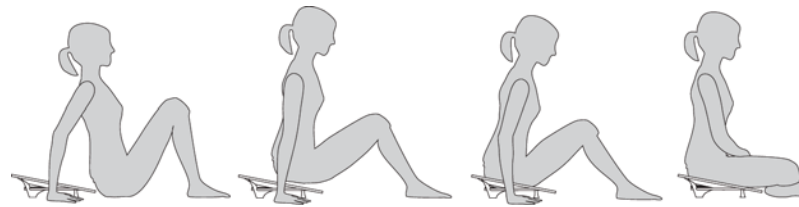

It is advised that you NOT slide too far back into the Seat, rock to the side or back, or stand on the chair as it will make the chair unstable. If you are uncomfortable or develop pain while sitting in the Seat, discontinue use of the chair and contact a medical professional. DO NOT stand or kneel on your Seat or use it other than as set forth in these instructions.

4 Care

Chair: Your Salubrion Seat is made of injection-molded plastic and may be cleaned using water and a soft cloth. Ensure that your Seat is completely dry before resuming use of it.

Cushion: The cushion included with your Salubrion Seat has a removable outer cover. Unzip the cover and remove the foam cushion core before cleaning. Gently hand wash the cover and air dry completely before reinserting the foam cushion core.

Salubrion, LLC Limited Warranty

This limited warranty extends to original consumer purchaser or any person receiving this product as a gift from the original consumer purchaser and to no other purchaser or transferee.

Limited One (1) Year Warranty

Salubrion warrants this product against defects in materials or workmanship for a period of one (1) year after the date of original retail purchase. During this period, Salubrion will repair or replace a defective product or part, at their discretion, with a new or refurbished product or part without charge to you. You must deliver the entire product to Salubrion. You are responsible for all transportation and insurance charges for the unit to and from the Salubrion's Service Center.

Your Responsibility

The above warranty is subject to the following conditions: (1) You must retain your bill of sale or provide other proof of purchase. (2) You must notify Salubrion within thirty (30) days after you discover a defective product or part. (3) All warranty servicing of this product must be made by Salubrion or a Salubrion Authorized Service Station. (4) Salubrion warranties are effective only if the product is purchased and used in the same continent territorial regions. (5) Warranties extend only to defects in materials or workmanship as limited above and do not extend to any product or parts which have been lost or discarded by you or to damage to products or parts caused by misuse, accident, damage caused by Acts of God, improper maintenance or use in violation of instructions furnished by us; or to products which have been altered or modified. (6) A physically damaged product is not acceptable for repair or exchange whether in or out of warranty and will be returned as received.